

The product



Thermogel comfort

An exclusive fastening system enables application to every part of the body. Ideal for sports injuries.

Code 00 012451 000 000

Thermogel comfort

10 x 10 cm

Useful for localised traumas and toothache. Suitable for children



Code 00 014351 000 000

Materials sales point



Display box



Stand display



ARTSANA S.p.A. - Via Saldarini Catelli, 1 - 22070 Grandate (CO) Italy
 Telefono 031.382.111 - Fax 031.382.400
www.picindolor.com



THERMOGEL

Comfort



The safe, comfortable
hot and cold therapy

The hot and cold therapy

An **effective**, irreplaceable technique for healthcare and sports **therapy**.

Cold therapy

Relieves painful conditions accompanied by swelling and inflammation:

- Relieves muscle spasm, traumatic **pain**, acute **inflammation** and can induce a mild local anaesthetic effect
- May be useful to control bleeding: **reduces local blood flow** by 50% after 10 minutes.

WHEN TO USE

- ❄ Recommended for small contusions, microtraumas (muscle strains, sprains and dislocations) and minor bleeding.
- ❄ Calms headache and toothache.
- ❄ Soothes localised skin inflammation.
- ❄ Indicated for insect stings.




Hot therapy

Heat is one of the oldest methods to relieve pain and treat contractions and spasms. The application of heated pads which release heat gradually over time can:

- Reduce joint stiffness
- Relax muscle spasms
- Provide an analgesic effect
- Improve blood circulation.

WHEN TO USE

- ☀ Recommended for lumbago, sciatica, muscle and abdominal pain.
- ☀ Indicated for stiff muscles and rheumatic muscle pain.

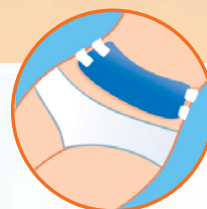



THERMOGEL

Comfort



The natural therapeutic power of hot and cold in a practical, versatile pack.



PRACTICAL

- **Easy to prepare**
COLD: Put Thermogel Comfort in the freezer for at least 2 hours.
HOT: Immerse in hot water (max. 60°C) for 10 minutes or heat in a bowl of water in the microwave (max 300 Watts) for 6 minutes.
- **REUSABLE.**
- The **extra soft fabric cover** reduces skin irritation.
- **Effective fastening** leaves the hands free.
- **Practical even during movement.**

VERSATILE

- **New fastening system**
- Thermogel comfort 10 x 26 cm is fitted with **two elasticated bands** for application to every part of the body.

